

## USER MANUAL - BODY BAGS

### General warnings

- Before using the body bag, the user must carefully read the attached instructions, paying particular attention to the correct safety measures and procedures to be followed for correct use.
- In the event of any abnormalities or damage to the body bag, the body bag must be removed from service immediately.
- Do not modify the body bag in any way.
- Ensure that all necessary precautions are taken to avoid the dangers which may arise from contact with blood or body fluids.
- The user must always wear gloves.
- In case of risk of contamination, wearing PPE is mandatory.
- Use a body bag only under the supervision of an experienced person.
- At least two users, one of whom must be experienced, are required to move the body.
- The body bag must not be exposed to or come into contact with sources of heat or combustion or flammable substances.
- Make sure there are no sharp objects in the immediate vicinity that could puncture the foil.
- The zip must always point upwards to avoid losing moisture and strength
- The body bag may not be reused under any circumstances.

### Transport and storage

- The pallets are not stackable.
- Keep the original packaging for storage and use in the event of further transport.
- The body bag must be stored in a dry place and away from sunlight.
- The ideal storage temperatures are between -25°C and 50°C.

### Usage

- For body bags without handles, use of a stretcher is mandatory. When the body is in the body bag, roll the body onto its side and slide the stretcher under the body.
- Under no circumstances should the body bag be picked up in any place other than the handles provided.
- Open the body bag lengthways and one side at a time (not the side where the body lies).
- Open the zip completely
- Place the body bag as close as possible to the body.
- Roll the body onto its side and slide the body bag under the body as far as possible.
- Roll the body back onto its back, partially inside the body bag.
- Now roll the body to the other side and open the body bag completely.
- When the body is rolled back, the zip can be closed.